# PORTI<sup>®</sup> 9

## SLEEP DOC

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**EN** Instructions for Use

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## Table of Contents

<b>1. Introduction.....</b>	<b>20</b>
1.1. Assigned Purpose.....	20
1.2. Contraindications.....	20
1.3. Intended users.....	20
1.4. Accessories.....	20
<b>2. Information.....</b>	<b>20</b>
2.1. Safety-related information.....	20
2.2. General information.....	21
2.3. Technical inspection.....	22
2.4. Liability for functioning or damage.....	22
<b>3. Recording measurement data.....</b>	<b>22</b>
3.1. Sensors.....	22
3.1.1. Sensors for flow and snoring noises.....	22
3.1.2. Thermistor sensor for breathing.....	22
3.1.3. Sensor for PAP (pressure).....	23
3.1.4. Sensor for oxygen saturation and pulse frequency .....	23
3.1.5. Sensor for respiratory effort.....	23
3.1.6. Sensor for body position.....	23
3.2. Preparing for measurement (Ambulatory).....	23
3.3. Operating elements of the device with display... 23	
3.3.1. Starting the measurement.....	23
3.3.2. Signal control.....	24
3.3.3. Stop the measurement.....	24
3.3.4. Battery indicator.....	24
3.3.5. Service menu.....	24
3.4. Operating elements of the device without display.....	24
3.5. Attaching the device unit and starting the measurement.....	24
3.6. Online Measurements (Inpatient).....	24
<b>4. Service and Maintaining the device.....</b>	<b>25</b>
4.1. Charging the battery.....	25
4.1.1. Charger.....	25
4.1.2. Charging Cradle.....	25
4.2. Cleaning instructions.....	25
4.3. Maintenance.....	25
4.4. Transport and storage conditions.....	25
<b>5. Evaluation software for PC.....</b>	<b>26</b>
5.1. Installing the software.....	26
5.2. Selecting a printer.....	26
<b>6. Troubleshooting.....</b>	<b>26</b>
<b>7. Technical specifications.....</b>	<b>27</b>
<b>8. Used Symbols.....</b>	<b>28</b>
<b>9. EMC Information.....</b>	<b>29</b>
<b>10. Index.....</b>	<b>32</b>

## 1. Introduction

### 1.1. Assigned Purpose

The Sleep Diagnosis System SleepDoc Porti 9 is a recording system for use in professional health care facility and home healthcare environment. The device is used for differentiated pre-diagnosis of sleep disorders and therapy control. It is intended for use by or on the order of physicians in adults and children.

The device allows the continuous recording of up to 12 measurement channels for at least 15 hours without data reduction. The following physiological variables are recorded:

- Flow
- Thermistor
- Oxygen saturation SpO<sub>2</sub>
- Pulse frequency
- Pulse wave
- Body position
- Snoring sounds (internal microphone)
- PAP (Positive Airway Pressure)
- Ambient Light
- Thorax effort
- Abdomen effort
- Activity

### 1.2. Contraindications

There are no absolute or relative contraindications to using the device. In the following cases, the device must be used under the supervision of qualified medical personnel:

- in patients with acute life-threatening illnesses.
- in patients with acute, serious infections.
- in mentally confused patients.
- in infants and children

### 1.3. Intended users

The Porti 9 sleep diagnosis system may only be used by authorized users. Authorized users include medical professionals who have been carefully trained in accordance with the instructions for use, such as:

- Physicians
- Medical professionals

### 1.4. Accessories

The device system as a whole (ME system) contains two components:

- The microprocessor controlled Porti 9 recording device (patient recorder)
- PC software for measurement data presentation and PC analysis

It is also equipped with the following applied parts:

- SpO<sub>2</sub> finger sensor with cable for obtaining pulse frequency and oxygen saturation values
- Flow prong for obtaining the respiratory signal.

- Adapter hose for measurement during PAP Therapy
- Flexible carry belt with integrated pressure transducers (effort sensor) to fasten the device to the patient and to record thoracic effort
- Flexible belt with integrated pressure transducers (effort sensor) to record abdominal effort
- Carrying console for patient recorder.

Other accessories:

- USB Interface cable for data transfer between the patient recorder and PC
- Battery Charger
- Shoulder bag for storing the device

optional applied parts:

- Thermistor for additional breathing detection

Other optional accessories:

- Battery cradle
- USB isolator for online measurement

The PC and the battery charger are not suitable for a domestic or patient environment due to lack of drip protection!

The data can be displayed and analysed on a standard commercial PC. The measurement curves and the analysis results can be printed on all commonly available printers, including dot-matrix, laser or ink jet printers.

The PC software analysis program OR5 has the following minimum configuration requirements

- PC with Windows operating system and i5 processor (i7 processor recommended)
- Microsoft Windows 10 or higher
- 8 GB of RAM (32 GB recommended)
- 500 GB of disk space on the hard drive
- Mouse
- Free USB port
- Printer with Windows driver

## 2. Information

### 2.1. Safety-related information

Observe the instruction manual:

Every use of the device requires exact knowledge and observance of this instruction manual. The device is only intended for the use described herein.

No alarm function available!

The device is not suitable for continuous monitoring of vital or physiological functions (for example, intensive monitoring, monitoring operation) since there is no SpO<sub>2</sub> alarm. No direct data analysis takes place in the device.

No SIDS Monitoring:

The device is not suitable for use as a SIDS monitor (SIDS: Sudden Infant Death Syndrome, sudden child death).

#### Patient instruction:

Patient instruction must be provided by a healthcare professional trained in the device. An attached short manual does not replace the manual instructions or the warning of possible hazards.

#### Separation from the supply network::

To isolate the device from the supply network, the plug of the power supply must be disconnected.

#### **Do not open the device!**

#### Warning:

Additional devices that are connected to medical electrical (ME) devices must be able to meet their IEC or ISO standards. Anyone connecting additional devices to ME devices is a system configurator and is responsible for the ME system being in compliance with the normative requirements (for example, IEC 60601-1).

#### Warning:

For online measuring you must use an electrical or optical isolator for connection of the device to the PC (available from your home care provider). A connection of the device to the PC without electrical or optical isolator is only allowed if all patient connections have been removed first. Only Physicians or trained staff is allowed to perform online measurements.

#### Warning:

Magnetic and electrical fields can impact on the functioning of the device. Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally. When operating the device, ensure that all third-party devices being operated in the vicinity are meeting their relevant EMC requirements. X-Ray equipment, HF surgical devices, tomography devices etc. can interfere with other devices as they may emit higher levels of electromagnetic interference.

#### Warning:

The device has no defibrillator protected application parts! Disconnect the device before defibrillation! A direct application of the device to the heart (esp. ECG!) is not allowed!

While the device is being worn, no invasive or intracorporal measurements and interventions (e.g., electro- or HF-surgical devices) may take place.

#### Warning:

Using multiple devices for patient screening could result in excessive leakage current above the allowed value!

#### Warning:

When using the device on patients with active implants, such as Cardiac pacemakers, peripheral nerve stimulation devices, tongue pacemaker etc.,

medical professionals should be alert to possible interference with the device or the implant.

#### Warning:

Applying the device to more than one patient at any one time is not allowed.

#### Warning:

The device must not be operated in vehicles and aircraft.

#### Warning:

When applying sensors to the patient, please ensure that no electrical line of the device is in contact with any other electrical conductive parts including ground.

#### Warning:

Avoid placing this instrument in direct sunlight or in close proximity to intense heat. Prevent also the contact with dust, lint, dirt, moisture and liquids.

#### Warning:

Children or incompetent persons should not use this device unattended without first having obtained instructions for the safe use of the device. Ensure that infants, children and animals cannot touch the device unattended

#### Warning:

Caution must be taken to ensure that cables do not encircle the patients neck. Special attention is needed in the case of children.

#### Warning:

Before the battery is charged, the charger and, if using the charging cradle, the battery must be checked for external damage!

#### Warning:

Before every application, the housing of the device as well as the cables and sensors must be checked for external damage!

#### Warning:

The device and PC software are not intended for automatic diagnosis only. The measurement data must be always reviewed and evaluated manually by a qualified physician or healthcare professionals.

## **2.2. General information**

This instruction manual should be regarded as a component of the device. It should be kept on hand somewhere near the device at all times. Reading and understanding the instruction manual is a prerequisite for proper use and correct handling of the device in order to maintain the safety of the patient and operator.

The guarantee is valid for a period of 24 months for the device and 6 months for the equipment from the date of sale.

Only accessories which are listed in this instruction manual should be used with the device. We cannot

guarantee the safe operation and function of the device if unknown proprietary accessories / consumables are used that have not been tested. (E.g. Patient leads, Sensors, Consumables etc.)

**Damages resulting from the use of third-party accessories or consumables shall render this warranty void.**

The manufacturer will only assume responsibility for the device in terms of safety, reliability and function if:

1. **assembly, add-ons, reinstallation, changes and repairs are carried out by the manufacturer or a qualified agent authorized by the manufacturer to do so;**
2. **The device is used in accordance with the instruction manual.**

All printed material relates to the model of the device and the safety regulations at the time of printing. All devices, switches, processors, software programs and names contained herein are subject to copyright law.

The manufacturer shall only be liable for the malfunction of the device and its software if used in the normal operating conditions in accordance to this manual.

Medical devices must only be operated by a qualified experienced person(s) to ensure correct handling of such devices.

The operator must read and understand the user manual to operate the device correctly.

The operator must check the functionality of the device before each use to ensure it is in sound condition and in good working order.

The user must report any serious incident related to the device to the manufacturer and to the competent authority of the Member State in which the user is established.

Functional testing of the device must be carried out at regular intervals. It is recommended that this is conducted once a month.

Upon reaching the end of its service life, the device and its accessories should be disposed of in accordance with the WEEE Directives or relevant Electronic Disposal protocol.

For further details please contact the manufacturer.

### **2.3. Technical inspection**

The device is maintenance-free. It is recommended to have a technical inspection carried out by the manufacturer or an agency authorized by the manufacturer every 24 months.

A functional tester can not be used to assess the accuracy of the finger sensor or pulse oximeter of the device.

The following tests need to be carried out by the operator before each measurement:

- Visual inspection of the device and accessories for obvious damage which could result in mechanical failure.
- Testing all hose connections are air tight.
- Checking the display LEDs.
- Testing the battery capacity.

### **2.4. Liability for functioning or damage**

In the event of improper use or repair by the owner or operator, the liability for the device will be transferred to those parties. The manufacturer shall not be liable for damages arising from non-observance of this instruction manual. The actual guarantee and warranty conditions in the manufacturers terms of sale/delivery are not extendable.

## **3. Recording measurement data**

### **3.1. Sensors**

#### **3.1.1. Sensors for flow and snoring noises**

The detection of patient flow is determined by the use of a nasal flow prong that transmits the pressure signal to an internal pressure transducer of the device. The flow prong can be easily applied by the user and will have no adverse affect on the quality of sleep. The use of a flow prong also eradicates the problem where adhesive sensors cannot be used. (i.e.: patients with beards). In addition the flow prong is an economical choice of sensor as it is commonly available. No additional sensor is required for respiratory and snoring detection. Respiratory sounds are transmitted to the device via the flow prong. The built in microphone detects the sounds which are then electronically analysed. Therefore additional microphones are not required. Due to the high sensitivity of the internal pressure transducer, the device is capable of measuring and recording ultra fine pressure differences associated with patients who mouth breathe.

The flow prong must be attached to the socket marked with the following symbol: 

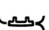
The flow prong should be used in accordance with the manufacturer's instructions.

#### **Warning:**

The flow prong is intended for single patient use. No attempt should be made to clean or use the flow prong the more than once. Multiple use can result in cross infection!

#### **3.1.2. Thermistor sensor for breathing**

Additional to the flow prong a Thermistor can be used to measure the breathing. Please also observe the instructions accompanying the sensor.

The Thermistor must be attached to the socket marked with the following symbol: 

### 3.1.3. Sensor for PAP (pressure)

Connection of PAP adapter: The PAP adapter is connected with the socket marked with 

### 3.1.4. Sensor for oxygen saturation and pulse frequency

A pulse oximeter has been integrated into the device for the purpose of measuring oxygen saturation and pulse frequency. A failure of the finger sensor or a missing data update by the pulse oximeter is indicated to the user by a red SpO<sub>2</sub> symbol on the display. At the same time the values for SpO<sub>2</sub> and pulse are fed to 0.

Please ensure that blood circulation at the point of measurement is not affected by the manner in which the sensor has been secured. Excessive pressure should not be exerted on the finger, especially if the temperature exceeds 41 ° C.

When using a finger sensor please secure the sensor and cable. All nail polish (coloured or clear) must be removed from the measuring finger to obtain usable data.

To minimize interference (e.g., motion artifacts), the pulse oximetry values are filtered by digital data processing. As a result of data averaging and signal processing, slight delays in the display of the pulse oximetric values occur. The internal pulse oximeter works with a time horizon of 4 seconds. By additionally considering the change tendency, the minimum value of saturation at the end of an apnea is correctly reproduced. The data is updated at every beat, so there are no measurable delays due to data refresh and transmission.

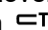
The finger sensor must be attached to the socket marked with the following symbol: 

Please read and understand the instructions accompanying the finger sensor.

### 3.1.5. Sensor for respiratory effort

The sensors used for recording thorax and abdominal movement consists of two small rubber pressure pads that are connected to the device via measuring hoses. The sensors are inserted into the pockets of the elasticated effort belts. The thorax belt is applied at the height of the sternum, the abdominal belt in the stomach region.

For hygiene reasons and to avoid allergic reactions the belts should be **worn over the clothes**.

The effort sensor to monitor thorax movement must be attached to the socket marked with  and the effort sensor to monitor abdominal movement must be attached to the socket marked with .

The effort belts are adjustable in size due to the Velcro fasteners, and should be suitable for most patients. These belts are however also available in special sizes.

### Note:

Over stretching of the belts may result in poor signals and possible loss of data.

### 3.1.6. Sensor for body position

The position sensor integrated in the device displays information related to the patient's body position during the study.

For accurate determination of body position, ensure the device is applied to the patient correctly. The body positions detected are: Supine, Prone, Left side, Right side, Upright.

## 3.2. Preparing for measurement (Ambulatory)

In preparation of an ambulatory study where the patient will use the device at home, please ensure the following:

1. Charge the battery: see (Page 25).
2. Start the PC software.
3. Fill out the fields relating to personal data of the patient in the menu item "Record / Initialize device" and start the transfer.
4. Connect the device with the USB interface cable. Socket: 
5. A dialogue window will be displayed in order for the operator to determine the start time and channel set for recording.
6. The device is now ready for use. The device can be disconnected from the PC.

**Note:** remove cable by pulling on the plug - not on the cable.

### 3.3. Operating elements of the device with display

Button:	Designation:
	On / off switch
	Arrow left
	Arrow right

The device will automatically switch on / off when connected / disconnected to a PC.

#### 3.3.1. Starting the measurement

The device has an internal clock for timer recording. If the timer has been programmed, recording will start automatically at the predetermined time. Up to eight recordings can be programmed.

**Note:** If necessary, the measurement can be started by the patient before the selected start time. To do this, the on / off switch must be pressed for approx. 2 seconds until the display switches on. The measurement can then be started manually by pressing the right arrow key. The measurement starts as soon as the countdown from 5 to 0 has occurred.

The duration of the measurement is now shown on the display.

During the measurement, the device's display switches off automatically while the measurement continues.

### 3.3.2. Signal control

After switching on the device with the on / off switch, the signals of the device (pulse oximetry, breathing movements, breathing) can be checked on the display from the start screen by pressing the left arrow key. The signal control is available both with and without running measurement.

### 3.3.3. Stop the measurement

A running measurement can be ended manually. To do this, keep the on / off switch  pressed until the symbol is completely red and then no longer visible in the display.

### 3.3.4. Battery indicator

When the display is switched off the charge status of the battery can be displayed by pressing the left or right arrow key.

### 3.3.5. Service menu

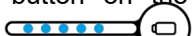
When the display is switched off the service menu can be displayed by pressing the left and right arrow keys at the same time.

**Note:** All of the above functions can only be carried out when the device is disconnected from the PC.

## 3.4. Operating elements of the device without display

The device will automatically switch on / off when connected / disconnected to a PC. The device has an internal clock for timer recording. If the timer has been programmed, recording will start automatically at the predetermined time. Up to eight recordings can be programmed.

Note: If necessary the device can be switched on by the patient before the selected start time by pressing and holding the Start button for one second. By pressing the start button (longer than 7 seconds), the device can be switched off by the patient.

The charging condition of the units battery can be displayed by pressing the button on the battery display on top of the device: 

Attention: This function will only work when the device is switched off and disconnected from the PC.

## 3.5. Attaching the device unit and starting the measurement

When applying the sensors, make sure that they are applied in such a way that strangulation is not possible due to their length (for example flow prongs, finger sensor). In order to ensure reliable recording

of data, you need to observe the following points when attaching the device and sensors. Before ambulatory measurements these steps should be demonstrated and practised with the patient beforehand in the clinic:

1. Secure the device to the thorax using the elasticated thorax belt (with the **two** built in pockets). Apply the strap whilst in the standing position and exhale. A correctly fitted strap should not slip up or down once fastened. To record abdominal breathing, attach the abdominal strap around the stomach in the same way. Always ensure the straps are fitted over the night clothes and not directly onto bare skin.
2. Check the position of the pressure transducers (black rubber pads). The sensors should be fully inserted into the built-in pockets sewn onto the strap. Never pull on the tubing connected to the pressure pads.
3. Apply the flow prong and/or respiratory thermometer to the nose or fix the adapter hose to the oxygen mask.
4. Apply the finger sensor (SpO<sub>2</sub>) to a finger and secure the cable to the wrist. Whilst doing so, ensure that you do not apply pressure to the finger thereby disturbing the blood flow.
5. The device will automatically switch itself on at the predetermined time.

Once the patient has been educated on fitting the device, the device can given to the patient to take home. The patient can refer to the quick reference guide when fitting the device at home.

### When going to bed the patient needs to:

- Attach the device using the elasticated carrying strap.
- Attach the flow prong and finger sensor.
- If applicable, attach the abdominal strap and all other sensors.
- Check that the sensors and their connectors are positioned correctly.

### The next morning:

- The patient has to detach the device and sensors. Cleaning of the sensors will be carried out by the technical staff at the clinic or hospital.
- Bring the device back to the clinic or hospital.
- For automatic evaluation the measurement can be transferred to the PC using the device software. Select the "Record" tab followed by "Read measurement from device".

## 3.6. Online Measurements (Inpatient)

Please adhere to the following steps when performing an online measurement:

1. Ensure the battery is fully charged (see Page 25).
2. Apply the device and connect all sensors to the patient (see Page 24)
3. Start the software on your PC.
4. Complete the Patient data fields in the menu item "Record / Start recording (Online)" and start the transfer.
5. Connect the device to the galvanic separation PC using the USB interface cable.
6. Once the test is complete, (the following morning) click the stop button  to end the measurement.
7. Detach the device from the USB connection

## 4. Service and Maintaining the device

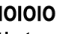
### 4.1. Charging the battery

**Caution: It is not allowed to use a different battery charger than the one provided! The charger must not be operated in the patient's environment!**

#### 4.1.1. Charger

A power supply unit is available for charging the battery. This is specially matched to the device's battery.

##### Battery charging procedure:

- Connect the battery charger to the device socket marked with the symbol 
- Plug the charger into a power outlet
- Device with display: The integrated LED's in the top cover of the device displays the charging status of the batteries.  
Device without display: The charging condition of the battery is displayed at the top of the device 

The device can remain connected to the charger adaptor for a long period of time without suffering any damage.

#### 4.1.2. Charging Cradle

To charge the battery optionally, the battery compartment on the back of the device can be opened and the battery removed. The battery charging adaptor is specially designed for this type of battery and must only be used with the device battery. It is recommended to charge the battery completely after each measurement.

##### Battery charging procedure:

- Remove the battery from the device and insert it into the battery cradle.
- Plug the charger into a power outlet
- The charger LED will illuminate orange. As soon as the battery is charged, the LED will illuminate green. The battery can remain in the charging

cradle for a longer period of time without damage.

### 4.2. Cleaning instructions

#### General

As with any medical device, when using the Porti 9 system, certain hygienic procedures are required for safe reuse on the patient. Reusable products must have safe disinfectability to exclude any risk of infection from subsequent users / patients. The regulations of the Medical Devices Act stipulate that for such medical devices disinfection activities with procedures according to the RKI guideline are to be carried out in analogy to the methods of surface or instrument disinfection. Sterilization of the products listed below is not necessary.

The Porti 9 system is a medical device that is in the patient's environment when used as intended, with direct contact with the hands of the staff and the patient. These products are therefore classified as so-called "non-critical medical products" in accordance with the guidelines for hospital hygiene and infection prevention.

The cleaning and disinfection instruction for the device can be found in the "**Hygiene Plan SleepDoc Porti**", which can be requested from authorized dealers.

#### Warning:

No liquid should enter the device or plug connections. The sensors must not be immersed in liquid. Before cleaning, close the pressure connections of the device with caps!

Disconnect the charging and data cable from the device before each cleaning.

#### **In the washing machine**

The thorax and abdominal straps can be washed in the washing machine at 60 °C (use a laundry net!). Let the straps dry in the air.

#### **Attention:**

- Do not use a dryer
- Remove thoracic and abdominal sensor beforehand

#### **Disposables**

Disposables must not be used multiple times or on different patients. Follow the guidelines for waste disposal in hospitals.

### 4.3. Maintenance

The device is maintenance-free. It is recommended to have a technical inspection carried out by the manufacturer or an agency authorized by the manufacturer every 24 months.

In order to maintain basic safety and essential performance, no special measures need to be taken with regard to EMC.

### 4.4. Transport and storage conditions

The ambient temperatures for transport and storage are between -25 °C and +70 °C, where:

- -25 °C and +5 °C without restrictions,
- + 5 °C to +35 °C with a relative humidity of up to 90 %, without condensation,
- > 35° C to 70 °C at a water vapor pressure of up to 50 hPa.

Condensation must always be avoided.

## 5. Evaluation software for PC

### 5.1. Installing the software

To install the PC software, place the supplied CD into the CD-ROM drive or insert the supplied flash drive into the USB port of your computer. The program will automatically run. If the program does not run automatically open the Explorer on your operating system and select the CD-ROM drive or flash drive. Once selected double click the Setup.exe file from the software.

You can check your computer and network (if applicable) for PC software versions that may have been previously installed. To ensure the correct up to date version is selected, open the software installed and click the Search button. The program will then automatically find and use the latest version installed.

Note: Alternatively you can update the software by selecting menu item Tools / Update from the software. (Only for updating the software).

After selecting the target drive the installation program will now automatically create the necessary directories and copy all the required files to the hard drive. The installation will automatically insert a new group containing an entry for the new software in the Programs folder of the Start Menu. In addition a shortcut on your Windows desktop will be created.

After successful installation, remove the CD or flash drive and store it in a safe place.

### 5.2. Selecting a printer

The measurement curves and reports can be printed on any commonly available type of printer (ink jet, dot-matrix or laser printer) that is already installed on your Windows system. The correct printer driver can be selected using the Windows Control Panel application.

## 6. Troubleshooting

**Channels (e.g. pressure) are missing in the display.**

*The channels are deactivated and therefore do not appear during "Test" and "Record".*

Check which channels are active for recording using the menu item Options / Channel settings.

**Channels are missing after loading a measurement.**

They were not recorded or have been hidden for measurement data display.

**Flow signal is missing during recording or follows the edge of the range.**

*The flow signal follows a straight flat line in the middle of the channel.*

Check the flow prong on the patient and the connection on the device. When measuring during PAP therapy, check the connection of the PAP adapter hose on the oxygen mask and on the device.

**Signal amplitudes are very small or do not appear.**

Check the corresponding sensors on the patient and their connections to the device. Whilst doing so, check that the tube hoses and the black pressure pads of the thorax and abdomen sensors are intact. A leaking sensor can cause inaccurate readings in the thorax channel. The pressure pads should be attached to the patient securely without over tightening the belt and the tube hoses should be routed in such a way that they cannot kink.

Check all cables and connectors on the device and the PC.

**Pulse oximeters not responding.**

*The channels for oxygen saturation and pulse frequency are registering 0% and 0 P/min respectively despite the patient being connected.*

First check that the finger sensor is seated correctly on the patient and remove any nail polish that may be present. A small red light should be present in the sensor when the finger is inserted. If the light doesn't switch on when the sensor is applied to the finger, then you need to check the connection at the device and any intermediate extensions.

**Printout not working.**

*The printer prints characters on the page but in no apparent format.*

The wrong printer or printer driver has been installed.

*The printer is not responding to the print command.*

Check the printer cable and the connections to the printer and PC. The printer should be switched on and operational, i.e. the control LED's on the printer should be lit.

**Cannot establish connection to the device**

*The USB cable is not connected correctly.*

Check the connection of the USB cable to the device and the PC.

**USB interface was deactivated**

The USB interface can be activated in the PC software. Select the "Device connection" tab followed by menu "Settings".

**Battery is discharged.**

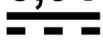
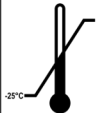
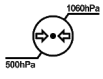
Charge the battery correctly.

**For clarification and in case of problems during the installation, maintenance or use, please refer to your local distribution partner, or directly to the manufacturer.**

## 7. Technical specifications

- Dimensions : 90 mm x 66 mm x 29 mm (L x W x H)
- Weight : 135 g including storage battery
- Housing : plastic (ABS, UL 94HB)
- Temperature range : +5 °C...+40 °C (operation)
- Moisture : 10 % - 90 % (operation)
- Atmospheric pressure : 70 kPa - 106 kPa (operation)
- Storage media : Internal SD card
- Storage capacity : min. 100 hours
- Registered parameters:
  - Respiratory activity : Differential pressure measurement via flow prong (with adaptor also during PAP therapy), alternative or additional measurement by means of thermistor possible
  - Thoracic effort : Differential pressure measurement by means of rubber cuffs built into chest strap
  - Abdominal effort : Differential pressure measurement by means of rubber cuffs built into abdomen strap
  - Breathing sounds : Phonometric transducer via flow prong
  - SpO<sub>2</sub>/Pulse : Built-in pulseoximeter, calibrated for functional oxygen saturation
    - SpO<sub>2</sub> measurement range: 80 % - 100 % ± 2 % SpO<sub>2</sub> 60 % - 79 % ± 4 % SpO<sub>2</sub>
    - Pulse measurement range: 50 1/min - 150 1/min ± 2%
    - (Reference electrical pulse simulator)
    - Finger sensor special rubber-coated thimble finger sensor
  - Pulse wave : Plethysmogramm display; measurement via finger sensor
  - Position : Acceleration sensor for position recording (5 positions)
  - Ambient Light : Photometric measurement and light-Intensity display
  - PAP : Different al pressure measurement directly on oxygen mask
    - Measurement range: 0 cmH<sub>2</sub>O - 45 cmH<sub>2</sub>O ± 5 %
  - Actimeter : Actimeter to record patient movement activity
- Fault indicator : LED / Display on top of the device
- Bluetooth-Interface :
  - Operation frequency : 2,45 GHz
  - Modulation : BT Low Energy 5
  - RF output power : 5 dBm max.
- Power supply : Rechargeable Li-Ion storage battery 3.6 V with built-in Semiconductor safety
- Charger : Plug-in power supply with medical approval
- Output : USB interface with cable for data transmission
- Power consumption : Approx. 85 mA
- Online operations : In online operation with a patient, a galvanic isolation to the PC is indispensable.

## 8. Used Symbols

Symbol	Meaning	Symbol	Meaning
	Pay attention to the instructions for use!		Electrical and electronic devices may not be disposed of with domestic waste. Consumers are obliged by law to return electrical and electronic devices at the end of their service lives to the public collection points set up for this purpose or point of sale. Details to this are defined by the national law of the respective country. This symbol on the product, the instruction manual or the package indicates that a product is subject to these regulations. By recycling, reusing the materials or other forms of utilising old devices, you are making an important contribution to protecting our environment.
	Medical device		
	Type BF		
	Protection class II		
	Best before date		
	Manufacturer	<b>IP22</b>	Device is protected against solid foreign bodies with diameter $\geq 12.5$ mm. Device is protected against access with a finger. The device is protected against falling dripping water when the housing is tilted up to $15^\circ$ .
	Manufacturing date		Once product! Neither suitable for a preparation nor for a multiple application.
	Catalog number		Serial number
	Unique Device Identifier	<b>5,9V</b> 	Maximum permitted charging voltage
	Temperature limit		Air pressure limitation

## 9. EMC Information

**Warning:** Medical electrical devices subjected to special preventive measures regarding to electromagnetic compatibility (EMC). They must be put in operation and installed according to the EMC information in this manual. Portable and mobile RF communications equipment (e.g. mobile phones) can influence medical electrical devices. Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result. For an intended use of the device, the specified equipment in this manual must be used. Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Guidelines and Manufacturer's Declaration – Electromagnetic Emissions		
The device is intended for use in an electromagnetic environment as specified below. The customer or user of the device should ensure that it is used in such an environment.		
Emission Test	Compliance	Electromagnetic Environment - Guidelines
RF – Emission according to CISPR 11 30 MHz - 1 GHz	Group 1	The device uses RF energy only for its internal function. So its RF emission is very low and it's implausible that it cause any interferences to surrounding electronic devices.  The device is suitable for the use in all establishments, including domestic establishments and those, which are connected to the public power supply, which powers buildings for residential purposes.
RF – Emission according to CISPR 11 30 MHz - 1 GHz	Class B	
Harmonic Emissions according to IEC 61000-3-2	Not applicable	
Voltage Fluctuations / Flicker Emissions according to IEC 61000-3-3	Not applicable	

Table 1: Table 201 EN 60601-1-2, Electromagnetic Emissions

Guidelines and Manufacturer's Declaration – Electromagnetic Immunity			
The device is intended for the electromagnetic environment as specified below. The customer and user of the device should ensure that it is used in such an environment.			
Immunity test	Test Specification	Compliance level	Electromagnetic Environment - Guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should consists of wood, concrete or should be covered with ceramic tiles. If the floor is covered with synthetic material, the relative humidity must be at least 30 %.
Electrical fast transient / burst IEC 61000-4-4	± 1 kV for input / output lines 100 kHz repetition frequency  ± 2 kV for power supply lines 100 kHz repetition frequency	± 1 kV  ± 2 kV	
Power Frequency (50/60 Hz) Magnetic Field IEC 61000-4-8	30 A/m	30 A/m	

Table 2: Table 4, 5, 7, 8 EN 60601-1-2, Electromagnetic Immunity

### Guidelines and Manufacturer's Declaration – Electromagnetic Immunity

The device is intended for an electromagnetic environment as specified below. The user of the device should make sure that is used in such an environment.

Immunity test	Test Specification	Compliance level	Electromagnetic Environment - Guidance
Conducted RF disturbance IEC 61000-4-6	3 V <sub>eff</sub> 150 kHz - 80 MHz 6 V <sub>eff</sub> ISM/Amateur bands 80 % AM / 1 kHz	3 Vrms outside the ISM band, 6 Vrms in the ISM and amateur radio bands  3 Vrms outside the ISM band, 6 Vrms in the ISM band	<p><b>Warning:</b> Interferences may occur in the environment of devices, which are marked with the following symbol:</p> 
Radiated RF disturbance IEC 61000-4-3	10 V/m 80 MHz - 2,7 GHz 80 % AM / 1 kHz	10 V/m 80 MHz to 2,7 GHz	
Proximity fields from RF wireless communications equipment IEC 61000-4-3	385 MHz (18 Hz Pulse Modulation (PM))	27 V/m	
	450 MHz (18 Hz PM)	28 V/m	
	710 MHz (217 Hz PM)	9 V/m	
	745 MHz (217 Hz PM)	9 V/m	
	780 MHz (217 Hz PM)	9 V/m	
	810 MHz (18 Hz PM)	28 V/m	
	870 MHz (18 Hz PM)	28 V/m	
	930 MHz (18 Hz PM)	28 V/m	
	1720 MHz (217 Hz PM)	28 V/m	
	1845 MHz (217 Hz PM)	28 V/m	
	1970 MHz (217 Hz PM)	28 V/m	
	2450 MHz (217 Hz PM)	28 V/m	
	5240 MHz (217 Hz PM)	9 V/m	
	5500 MHz (217 Hz PM)	9 V/m	
	5785 MHz (217 Hz PM)	9 V/m	
Proximity Magnetic Fields IEC 61000-4-39	30 kHz, CW, 8 A/m  134,2 kHz, 50 % pulse at 2,1 kHz, 65 A/m  13,56 MHz, 50 % pulse at 50 kHz, 7,5 A/m	30 kHz, CW, 8 A/m  134,2 kHz, 50 % pulse at 2,1 kHz, 65 A/m  13,56 MHz, 50 % pulse at 50 kHz, 7,5 A/m	

Table 3: Table 4, 5, 7, 8 EN 60601-1-2, Electromagnetic Immunity

Guidelines and Manufacturer's Declaration – Electromagnetic Immunity						
Test frequency MHz	Frequency band <sup>a</sup> MHz	Radio service <sup>a</sup>	Modulation <sup>b</sup>	Max. Performance W	Distance m	Immunity test level V/m
385	380 - 390	TETRA 400	Pulse Modulation <sup>b</sup> 18 Hz	1,8	0,3	27
450	430 - 470	GMRS 460, FRS 460	FM <sup>c</sup> ± 5 kHz Hub 1 kHz Sinus	2	0,3	28
710	704 - 787	LTE Band 13, 17	Pulse Modulation <sup>b</sup> 27 Hz	0,2	0,3	9
745						
780						
810	800 - 960	GSM 800/900 TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse Modulation <sup>b</sup> 18 Hz	2	0,3	28
870						
930						
1 720	1 700 - 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse Modulation <sup>b</sup> 217 Hz	2	0,3	28
1 845						
1 970						
2 450	2 400 - 2 570	Bluetooth, WLAN 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse Modulation <sup>b</sup> 217 Hz	2	0,3	28
5 240	5 100 - 5 800	WLAN 802.11 a/n	Pulse Modulation <sup>b</sup> 217 Hz	0,2	0,3	9
5 500						
5 785						
NOTE: If necessary, to achieve immunity test levels, the distance between the transmit antenna and the device can be reduced to 1 m. The 1 m test distance is permitted according to IEC 61000-4-3.						
a	The carrier must be modulated with a square wave signal with 50% duty cycle.					
b	The carrier must be modulated with a square wave signal with 50% duty cycle.					
c	As an alternative to Frequency Modulation (FM), pulse modulation at 50% duty cycle at 18 Hz may be used, as this, if not the actual modulation, would be the worst case scenario					

Table 4: Table 9 EN 60601-1-2, Test specifications for the immunity of enclosures to high-frequency wireless communication devices

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*The Sleep Diagnosis System is a recording system for use in  
 professional health care facility and home healthcare environment.  
 The device is used for differentiated pre-diagnosis of sleep disorders  
 and therapy control. It is intended for use by or on the order of  
 physicians in adults and children.*

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: IIa / Regel 10, 3. Spiegelstrich  
IIa / Rule 10, third indent

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Furthermore, the following applicable essential requirements of the following EC-Directives will be met:

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Directive 2014/53/EU on the harmonization of the laws of the Member States relating to the making available on the market of radio equipment and repealing Directive 1999/5/EC.

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## Ordering information (EN)

### Art.-No. Spare parts / consumer material / Accessories

930300	<b>Finger sensor premium SpO2</b> finger cuff with cable for long time measurement, M 9/10
929100	<b>Finger sensor premium SpO2, adults, with Hirose plug</b> , M 9/10
925305	<b>Velcro wrist strap</b> to fix finger sensor cable to the wrist (1 PU = 5 pieces)
920340	<b>Nasal Cannula</b> nose spectacles for flow measurement
NC250	<b>Nasal Cannula</b> nose spectacles for flow measurement
200-0312/10	<b>Flow prong</b> nose spectacles for flow measurement
929320	<b>Thermistor</b> with connection for flow prong, reusable, M 9
920307	<b>Flow prong with O2-connector</b> nose spectacles for flow measurement during oxygen therapy
920309	<b>Flow prong Nasal / Oral</b> with separate mouth tube
925220	<b>PAP adapter hose</b> for oxygen mask
920221	<b>Nipple</b> for PAP mask (1 PU = 10 pieces)
925235/TA80	<b>CPAP Adapter incl. PVC tube, Luer-Lock, 80 cm</b> , disposable
925236/TA200	<b>CPAP Adapter incl. PVC tube, Luer-Lock, 200 cm</b> , disposable
925233	<b>PAP Adapter hose</b> for oxygen mask Respiroics ®
929250	<b>Carrying console patient recorder</b> , M 9
929302	<b>Shoulder bag, grey</b> cloth bag with shoulder strap for carrying device, M 9
929305	<b>Shoulder bag, blue</b> cloth bag with shoulder strap for carrying device, M 9
929140	<b>Thorax sensor, complete</b> 2 pressure pads, flexible connection tubes, M 9
925395	<b>Abdomen sensor, complete</b> 1 pressure pad, connection hose, M 5/6/7/8/9
926281	<b>Thorax strap, flexible, blue, Size S</b> , M 5/6/7/9
926280	<b>Thorax strap, flexible, black, Size M</b> , M 5/6/7/9
926282	<b>Thorax strap, flexible, red, Size L</b> , M 5/6/7/9
926286	<b>Thorax strap, flexible, green, Size XL</b> , M 5/6/7/9
925388	<b>Abdomen strap, flexible, blue, Size S</b> , M 5/6/7/9
925389	<b>Abdomen strap, flexible, black, Size M</b> , M 5/6/7/9
925390	<b>Abdomen strap, flexible, red, Size L</b> , M 5/6/7/9
925391	<b>Abdomen strap, flexible, green, Size XL</b> , M 5/6/7/9
930224	<b>USB cable</b> for data transmission between device and PC, M 9/10
930230	<b>Electrical isolator USB</b> for galvanic separation between device and PC during online measurement; complete with adapter, M 9/10
929180	<b>Li-Ion Akku Patient Recorder</b> , M 9
929181	<b>Charging cradle for Li-Ion battery</b> , M 9
930209	<b>Power supply, medical, for Patient Recorder</b> , M 9/10
929330	<b>Cleaning kit</b> , M 9
929301	<b>Carrying case grey, with inlay</b> , for unit and accessories, M 9
929340	<b>Transport bag for Sleep Diagnostic System</b> , M 9

### Art.-No. Spare parts / consumer material / Accessories

929314	<b>Patient instructions Porti 9, EN</b>
929381	<b>Patient instructions Samoa, EN</b>
929321	<b>Patient instructions VitalNight PG, EN</b>
929318	<b>Instruction For Use Porti 9, GE-EN-FR-IT</b>
929310	<b>Instruction For Use Samoa, GE-EN-FR-IT</b>
929319	<b>Instruction For Use VitalNight PG, GE-EN-FR-IT</b>
925306	<b>PC-Software OR5</b>
925309	<b>PC-Software MSV</b>
925310	<b>PC-Software EasyScore</b>

### Art.-No. Spare Parts / Accessories for use with children

929150	<b>Thorax sensor, complete</b> , 2 pressure pads with flexible connection tubes and lengthened tube for positioning device near child, M 9
925396	<b>Abdomen sensor, complete</b> , 1 pressure pad with flexible connection tube and lengthened tube, M 5/6/7/9
925285	<b>Thorax strap for children</b> , flexible, black, <b>Size XS</b>
925286	<b>Thorax strap for children</b> , flexible, black, <b>Size XXS</b>
925287	<b>Thorax strap for children</b> , flexible, black, <b>Size XXXS</b>
925393	<b>Abdomen strap for children</b> , flexible, black, <b>Size XS</b>
925397	<b>Abdomen strap for children</b> , flexible, black, <b>Size XXS</b>
925398	<b>Abdomen strap for children</b> , flexible, black, <b>Size XXXS</b>
920126	<b>Infant sensor Philips SpO2</b> with Philips connector (has to be extended with Hirose- Philips extension 920122), <b>children 1 - 4 kg</b>
920127	<b>Infant sensor SpO2</b> with DSUB connector (has to be extended with Hirose-DSUB extension 920123), <b>children 4 - 40 kg</b>
920122	<b>Extension Hirose - Philips</b> for device and infant sensor, M 9/10
920123	<b>Extension Hirose - DSUB</b> for device and infant sensor, M 9/10

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Deutschland GmbH